


3.30"



MAKERS OF RESPIRATORY VALVES SINCE 1938

# HRI

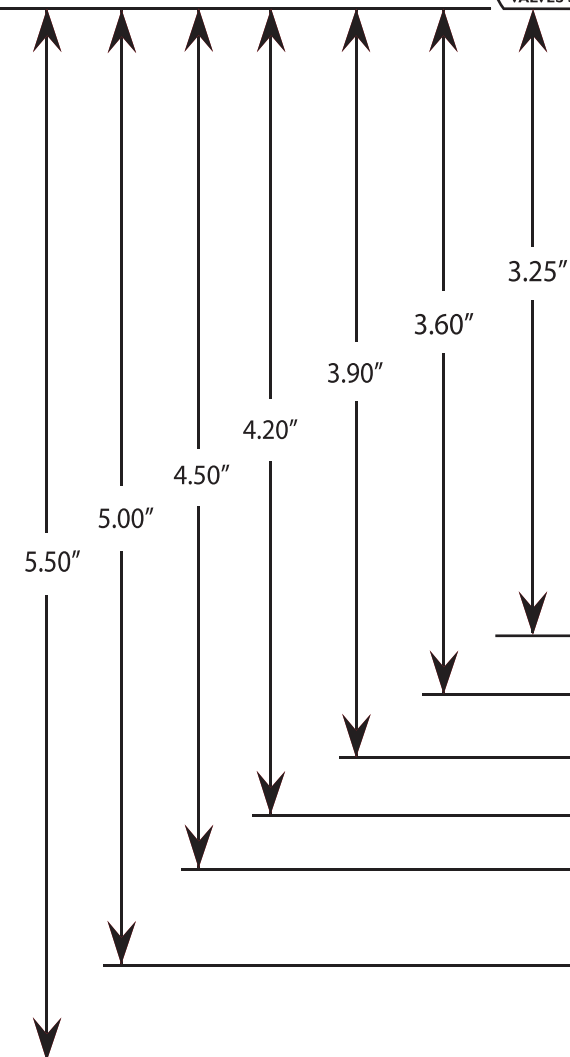

## MASK SIZING GAUGE

For Hans Rudolph, inc.  
Oro-Nasal Vmask™ and V2mask™  
Series Face Masks

**Directions:** Length measurements are used to determine the size of the Hans Rudolph Oro-Nasal mask that will best fit the patient. For measurement: the patient's facial muscles should be relaxed and jaw closed. Place the Gauge Tip to the deepest point of the nasal root: depression (sellion) and note the size indication at the bottom tip of the chin. The color band widths indicate the size ranges. (This gauge is a reference to aid in sizing and not an exact measurement as other factors may need to be taken into account.)

3.25"

3.60"

3.90"

4.20"

4.50"

5.00"

5.50"

7.25"

\*If between sizes, we recommend you use the smaller size.\*



**PETITE**

**EXTRA-SMALL**

**SMALL**

**MEDIUM**

**LARGE**

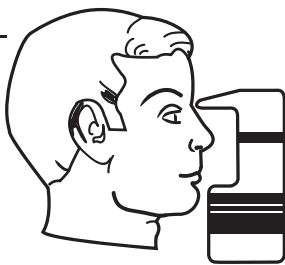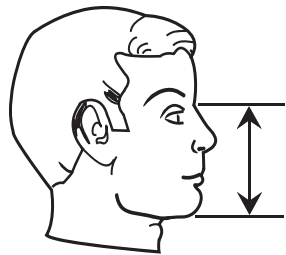

**HANS RUDOLPH, inc.**  
MAKERS OF RESPIRATORY VALVES SINCE 1938

**ISO 9001 ISO 13485**

Tel: (913) 422-7788 USA & CANADA (800) 456-6695

Fax: (913) 422-3337 E-mail: hri@rudolphkc.com  
www.rudolphkc.com

8325 Cole Pkwy, Shawnee, Kansas 66227 USA



Printed in USA  
691143 5/06 Rev B

2.75"